**VULPES GOODS**  
PETS

# **CHIP READER**

**MANUAL • ANLEITUNG  
HANDLEIDING • MANUEL  
MANUAL • MANUALE • MANUAL**

[WWW.VULPESGOODS.COM](http://WWW.VULPESGOODS.COM)

|                                  |           |
|----------------------------------|-----------|
| <b>ENGLISH</b>                   | <b>4</b>  |
| The Device                       | 5         |
| User Guide                       | 6         |
| Characteristics and Application  | 9         |
| Product parameters               | 10        |
| <b>DEUTSCH</b>                   | <b>11</b> |
| Das Gerät                        | 12        |
| Benutzerhandbuch                 | 13        |
| Eigenschaften und Anwendung      | 16        |
| Produkt parameter                | 17        |
| <b>NEDERLANDS</b>                | <b>18</b> |
| Het apparaat                     | 19        |
| Gebruiksaanwijzing               | 20        |
| Kenmerken en Toepassingen        | 23        |
| Product Parameters               | 24        |
| <b>FRANCAIS</b>                  | <b>25</b> |
| Le dispositif                    | 26        |
| Manuel de l'utilisateur          | 27        |
| Caractéristiques et applications | 30        |
| Paramètres du produit            | 31        |

## **ESPAÑOL**

**32**

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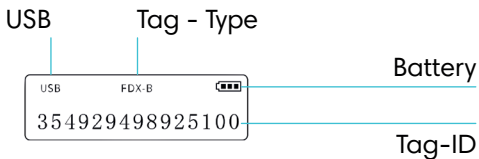
# THE DEVICE



## USER GUIDE

Press the scan button to turn on the device and start the scanning model.




Connect to Bluetooth. The name of the reader is “Top Group” Reader, locate and connect the chip reader.

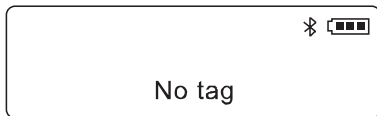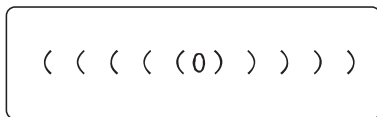If the connection is successful, the Bluetooth logo appears in the upper right corner of the screen, as shown in the figure.


If a tag is detected, the tag number is displayed on the screen.

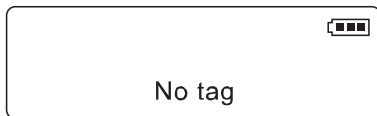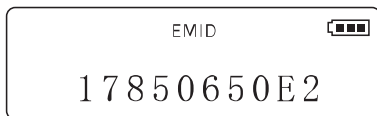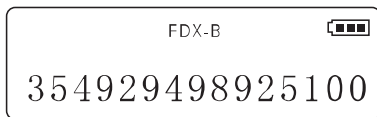
If no tag is detected, “No tag ” is displayed.

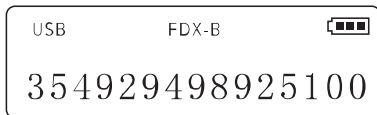The device can be charged and upload data via the USB cable. When the device is connected via USB, "USB" is displayed in the upper left corner and in charging status for battery status.

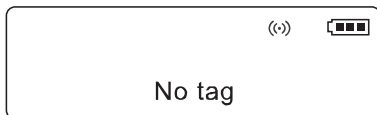The tag data will be uploaded to the receiving device in real time, and the wireless connection symbol will be displayed in the upper right corner of the screen after inserting the receiver.

Press the button for 3 seconds and the data will be uploaded from the scanner. Maximum of 128 records. When connected to USB, the device can upload data and scan tag together.

## **CHARACTERISTICS AND APPLICATION**

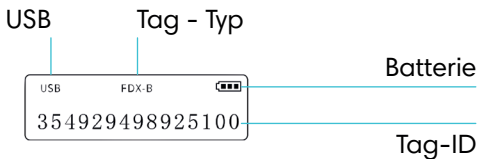
- Always use the correct USB cable for the connection.
- Keep the chip reader out of the reach of children. The product is not a toy.
- Keep the chip reader out of reach of excessive moisture and water. Do not immerse the chip reader also do not immerse it in water.
- Do not drop the chip reader from great heights. Drop damage can cause irreparable damage.
- Choose the right system for optimal operation. This is important for both your phone and PC.

## PRODUCT PARAMETERS

|                        |                          |
|------------------------|--------------------------|
| Model:                 | VG-290B                  |
| Frequency:             | 134.2Khz/125Khz          |
| Reading time:          | <100ms                   |
| Battery:               | 3.7 Lithium              |
| Memory:                | 128 chips                |
| Connection:            | USB 2.0, 2.4g, bluetooth |
| Operating Temperature: | -5 °C ~40 °C             |
| Storage Temperature:   | -5 °C ~40 °C             |
| Weight:                | 120g                     |

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